



GREEN&CLEAN M2

Disinfection of the Suction System

GREEN&CLEAN M2



1
Press dispenser pump twice
(= 6 ml of concentrate)



2
Fill container with cold water up to the indicator
(= 600 ml of 1% dilution)



3
Hold small suction tube on opening no. 1 until air is sucked in



4
Hold large suction tube on opening no. 2 until air is sucked in



5
Empty the remaining solution into the cuspidor

Disinfection with GREEN&CLEAN M2:

- 2 x daily (noon/evening) and after surgical treatments
- After every treatment, rinse the suction system with cold water
- Change colour after 80 applications (1 pouch)
- Wear protective gloves when disinfecting
- Exactly follow dosage directions

GREEN&CLEAN M2: Material friendly disinfection of the suction system

(including cuspidor, amalgam separator and air-water separation)

- dual system (red/green) against microbiological resistance
- HBV/HIV/HCV, bactericide/fungicide, tuberculocide
- highly concentrated and easy to dose
- excellent deodorising and protein dissolving power
- actively defoaming
- DGHM/VAH listed, CE 0408